


Sign In to Your Account

To sign in to your account:

1. Navigate to t:connect's home page by following the link below:

<http://tconnect.tandemdiabetes.com>

2. Enter the email address and password you used when creating your t:connect account.

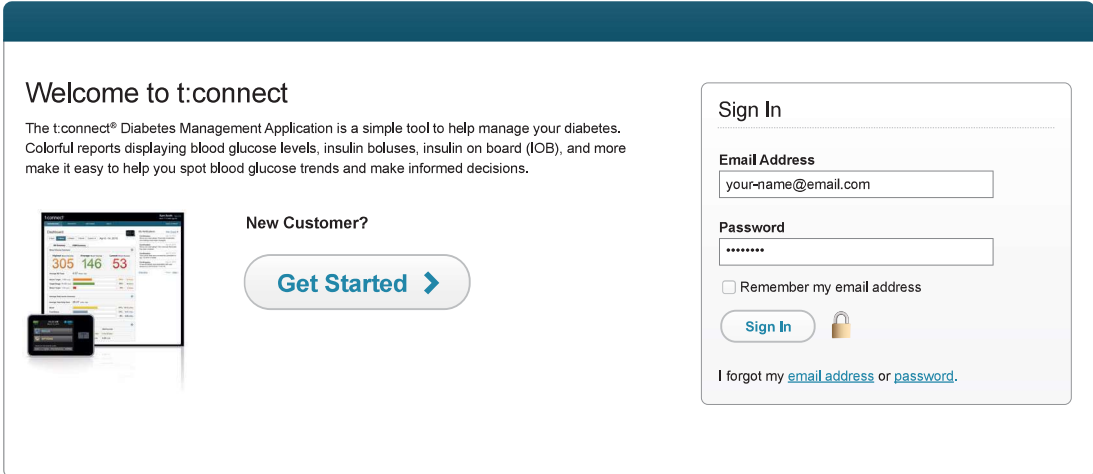

The screenshot shows the t:connect home page. On the left, there is a 'Welcome to t:connect' section with a brief description of the application and a 'New Customer?' button labeled 'Get Started >'. On the right, there is a 'Sign In' form with fields for 'Email Address' (containing 'your-name@email.com') and 'Password' (masked with dots). Below the password field is a checkbox for 'Remember my email address' and a 'Sign In' button with a lock icon. At the bottom of the sign-in form, there is a link: 'I forgot my [email address](#) or [password](#).'

NOTE


If you forget the email or password for your t:connect account, use the "I forgot my email address or password" links on the home page for instructions on how to get them.

Upload Data

To upload data from your pump or BG meter to your t:connect account, follow these steps:

1. Plug your pump or BG meter in to your computer.
2. Open the t:connect Uploader and click **Start Upload**.


Upload times will vary depending on the number of records since your last upload.

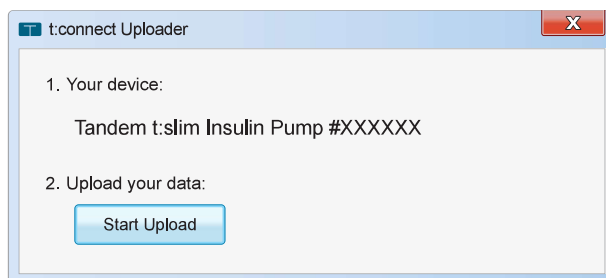

NOTE

You can also open the t:connect Uploader before connecting your device.

3. The “Upload Successful” message is displayed after the data has successfully been transmitted.

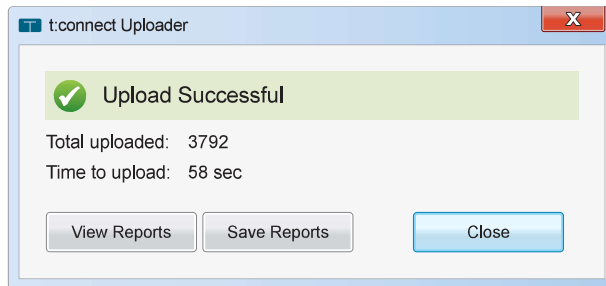

4. Click **View Reports** or **Save Reports** to be taken directly to the t:connect Application.

NOTE

You can also navigate directly to the t:connect Application through a web browser on any computer by going to <http://tconnect.tandemdiabetes.com>
