

# DR. JOEL KAPLAN



## RETURN POLICY

### Damaged Equipment

During the first year from date of purchase there is a one-time exchange for items damaged due to defect or shipping. The item(s) must be sent back to us at the below address accompanied by a photocopy of the original invoice and a brief note describing what happened. Be sure to include your address and a contact phone. Once we receive the damaged item we will promptly mail out a replacement.

### Wrong Size Cylinders

We cannot stress enough that when ordering it is imperative that you give your accurate measurements so we can send the correct size cylinder. During the first 30 days from date of purchase we will replace any incorrectly sized cylinder due to our error. The cylinder must be sent back to us at the below address accompanied by a photocopy of the original invoice, a note with your exact measurements and if you need a larger or smaller cylinder. Be sure to include your address and a contact phone. Once we receive the cylinder we will promptly mail out a replacement, simply follow the above procedure and include a payment of \$18 to cover shipping and processing. After 30 days we can offer you cylinders at a discounted price.

### Returning Complete Systems For Refund

During the first 30 days from date of purchase if you are dissatisfied with the system you can return the product for a refund less the shipping cost and a 10% restocking fee. All items must be cleaned and sent back to us at the below address accompanied by a photocopy of the original invoice and a note explaining your reason for the return.

*There are no refunds on  
BOOKS, VIDEOS, NOVELTIES, OR SUPPLEMENTS.  
All sales are final.*

### Warranties

All of our pumps come with a specified warranty at the time of purchase. Extended warranties can be purchased within 30 days from the date of purchase.

### Mail All Returns To:

Dr. Joel Kaplan, Inc. 1220 Rosecrans St., Suite 128, San Diego, CA 92106

There are no refunds on the cost of shipping.  
There is a 10% restocking fee for all returned items.

## STEP-BY-STEP GUIDE TO USING YOUR PUMP

DR. KAPLAN'S ELECTRIC PENIS PUMP SYSTEM MAKES PENIS PUMPING EASY & ENJOYABLE.

### GETTING TO KNOW YOUR PUMP

#### FIRST EXPANSION CYLINDER

The first expansion cylinder is made of medical grade acrylic and is especially made to your current size to allow for maximum growth.



#### ELECTRIC PUMP

The electric pump will be used to pump air out of the expansion cylinder to create a vacuum.



#### GAUGE AND HOUSING

The gauge and housing is used to connect the electric pump to the cylinder.

Using the pressure gauge, you can increase and decrease the pressure for your pumping session.



[www.DrJoelKaplan.com](http://www.DrJoelKaplan.com)

## HOW TO OPERATE YOUR ELECTRIC PUMP


### STEP 1: CONNECT THE HOISING TO THE PUMP

Your Electric Pump includes a black connector to fit your gauge and hosing. Push one end of the hosing into the black connector as shown:




### STEP 2: CONNECT TO YOUR CYLINDER

Insert the other end of the hose with the white connector into the connector on top of your cylinder.

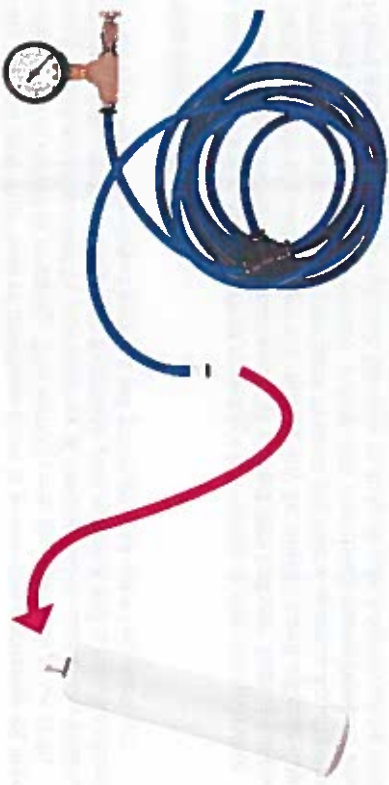

### STEP 3: PLUG AND GO

Turn on the Electric Pump by plugging in the outlet and flipping the red power switch in the front of the pump. Place the cylinder over your erect penis as shown below.

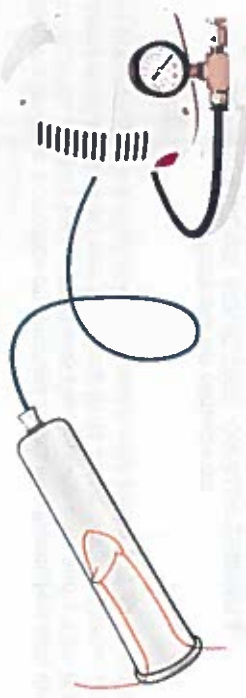

### STEP 6: CONTROL YOUR PRESSURE

Rotate the valve on your pressure gauge as below to increase and decrease the pressure in your cylinder. With the gauge facing you, turn the valve toward you to increase the pressure, and turn the valve away from you to decrease the pressure.



DECREASE  
PRESSURE

INCREASE  
PRESSURE