


LOSE WEIGHT AND KEEP IT OFF WITHOUT SURGERY

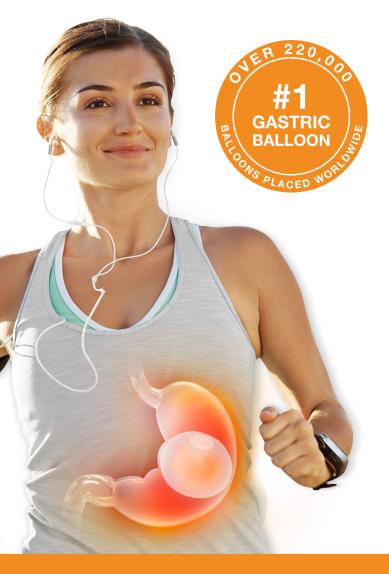

ORBERA® helps patients lose 3x the weight of diet and exercise alone.

START LOSING 3X THE WEIGHT WITH ORBERA®

TIRED OF SPENDING MONEY ON WEIGHT LOSS PROGRAMS THAT DON'T WORK?

The ORBERA® weight loss system provides you with drastic results in only 6 months - on average patients lost 3x the weight of diet and exercise alone.

ORBERA® combines the leading gastric balloon technology along with a support team of experts and a personalized diet and exercise plan all in one program.

This comprehensive program combines the portion control of the stomach balloon with the long-lasting healthy lifestyle coaching to provide effective and safe weight loss.



HOW ORBERA®

WORKS

THE 100% NON-SURGICAL SOLUTION



Deflated ORBERA® balloon is inserted through your mouth and into your stomach.



The balloon is then filled with a sterile saline solution to the size of a grapefruit.



The balloon takes up enough space in your stomach to help you adapt to healthier portion sizes.

6 MONTHS LATER



The balloon is removed 6 months later with a non-surgical procedure.

ORBERA® HELPS YOU FEEL FULLER AND EAT SMALLER MEALS.



FDA APPROVED SAFE AND EFFECTIVE

The ORBERA® weight loss system helps patients lose 3x the weight of diet and exercise alone.



NON-SURGICAL WEIGHT LOSS

ORBERA® is a reversible non-surgical weight loss procedure, which means no incisions, stitches, or scars.



REAL WORLD RESULTS

Millions of pounds have been lost with ORBERA®, with over 220,000 gastric balloons distributed worldwide.



#1 WEIGHT LOSS GASTRIC BALLOON

ORBERA® is the global leading weight loss balloon and has helped thousands meet their weight loss goals for over 20 years.





CHANGE YOUR LIFESTYLE WITH ORBERA® COACH

With ORBERA® Coach, you receive the personalized support and motivation you need to achieve your weight loss goals.

ORBERA® Coach is an innovative program designed to provide post-procedure support for patients during their weight loss journey. The 12 months of coaching includes live 1:1 and group sessions with registered dietitian coaches, comprehensive information, and motivational tools delivered via the digital device of your choice.





NOTE: Not all practices offer ORBERA® Coach. Some practices may offer their own registered dietitian and coaching support.

ORBERA® Coach provides you with:

12 Monthly 1:1 sessions with a registered dietitian

12 Monthly group sessions

100's of ORBERA® appropriate recipes

Mobile Apps: iOS and Android

Unique picture based food diary





YOUR QUESTIONS ANSWERED

How much does it cost?

The total cost of the ORBERA® Program will vary but is typically an all inclusive price that includes the ORBERA® Balloon, Placement Procedure, Removal Procedure, Facility Fees and 12 months of physician and dietary coaching support.

Some patients are able to save up to 31% of the total cost by using their tax free savings accounts (HSA / FSA). Also most practices may offer financing options for low monthly payments.

What does it feel like?

ORBERA® takes up space in patients' stomachs and it will make you feel very full with a small meal. It helps you learn proper portion control over the six months with it in your stomach.

It typically takes a few days for your body to adjust to the ORBERA® balloon and most patients experience some nausea, vomiting and reflux, but this typically goes away and is managed by your physician.

Does it really work? How much can I expect to lose?


ORBERA® has helped thousands of patients around the world lose that stubborn fat. ORBERA® has been studied in over 200 clinical studies around the world. In the FDA approval study, patients on average lost 3x the weight of diet or exercise, or about 10% of their total body weight.

When will I see the results?

The weight loss is rapid with most of the weight loss occuring in the first three months after balloon placement. While most of the weight loss occurs in the first few months, ORBERA® is a 12 month program to provide you the coaching and training so that you learn how to keep the weight off for the long term.

What do I do if I gain weight?

Many people are able to use the time with their ORBERA® Balloon and coaching to develop a new lifestyle so that they are able to maintain their weight loss or possibly continue to lose weight. This is why the coaching and follow up after the balloon removal is so important. However, some patients may regain weight after the balloon is removed. In the US clinical study, patients on average maintained ~75% of their weight loss 6 months after the balloon was removed.

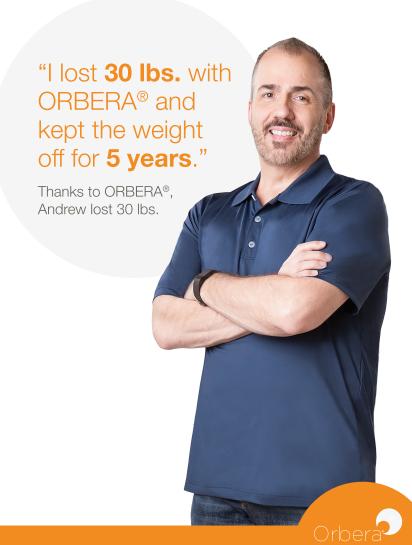

SUCCESSWITH ORBERA®



"I've had the ORBERA® balloon for five months now and I've lost 42 lbs."

Matthew, lost 42 lbs.*

"You'll have the support of your doctor, where you can learn how to change your whole life around."

Jerri, lost 40 lbs.*



"I feel great!





ORBERA® IN THE MEDIA











ARE YOU READY TO START LOSING WEIGHT WITH ORBERA®?

GET STARTED TODAY.

www.orbera.com

Important ORBERA® Intragastric Balloon System Safety Information:

The ORBERA® Intragastric Balloon System is a weight loss aid for adults suffering from obesity, with a body mass index (BMI) ≥ 0 and ≤ 40 kg/m2, who have tried other weight loss programs, such as following supervised diet, exercise, and behavior modification programs, but who were unable to lose weight and keep it off.

To receive ORBERA® you must be willing to also follow a 12-month program, beginning with the placement of ORBERA® and continuing for 6 months after, that includes a healthy diet and exercise plan. If the diet and exercise program is not followed, you will not experience significant weight loss results; in fact, you may not experience any weight loss.

Losing weight and keeping it off is not easy, so you will be supervised throughout this program by a team of physicians, physiologists, and nutritionists. This team will help you make and maintain major changes in your eating and exercise habits.

ORBERA® is placed for no more than six months. Any time that the balloon is in the stomach for longer than six months puts you at risk for complications, such as bowel obstruction, which can be fatal.

Some patients are ineligible to receive ORBERA®. Your doctor will ask you about your medical history and will also perform a physical examination to determine your eligibility for the device. Additionally, at the time of placement, the doctor may identify internal factors, such as stomach irritation or ulcers, which may prevent you from receiving ORBERA®.

You must not receive ORBERA® if you are pregnant, a woman planning to become pregnant within six months' time, or breast-feeding.

Complications that may result from the use of ORBERA® include the risks associated with any endoscopic procedure and those associated with the medications and methods used in this procedure, as well as your ability to tolerate a foreign object placed in your stomach. Possible complications include: partial or complete blockage of the bowel by the balloon, insufficient or no weight loss, adverse health consequences resulting from weight loss, stomach discomfort, continuing nausea and vomiting, abdominal or back pain, acid reflux, influence on digestion of food, blockage of food entering the stomach, bacterial growth in the fluid filling the balloon which can lead to infection, injury to the lining of the digestive tract, stomach or esophagus, and balloon deflation.

Important: For full safety information please visit orbera.com/dfu, talk with your doctor, or call Apollo Customer Support at 855-700-0780.

CAUTION: Bx only.

*Results are from individual customer testimonials and your results may vary.

MKT-01133-00 R01