



Naturally *Restylane*®

With the *Restylane*® line of products, your specialist can customize the ideal treatment plan to enhance your natural look.

Actual patient.  
Individual results may vary.

Please see Important Safety Information on pages 10-11 for the *Restylane* family of products.

# The way we age

## Natural aging affects the whole face

As we age, different parts of the face lose shape and fullness—leading to lines, wrinkles, and a less youthful appearance. In many cases, this is because our bodies aren't producing as much hyaluronic acid (HA) as they used to. HA attracts moisture, and less of it means that our features begin to sag and droop.<sup>1</sup>

Before



## Where we see wrinkles (and more)

**Cheeks (1)**—Loss of collagen, thinning of underlying fat, and decreasing bone volume can lead to a reduction of volume in the cheeks, giving a sunken appearance.

**Nasolabial Folds (smile lines) (2)**—Through volume loss and aging over time, the skin of our cheeks may drape over to become nasolabial folds (smile lines), creating a tired looking face with wrinkles between the nose and mouth.

**Lips (3)**—As we age, our lips may lose shape and there can be an increase in small vertical lines around the mouth (4—perioral lines and 5—marionette lines), while the corners sag (6—oral commissures).

Please see Important Safety Information on pages 10-11 for the *Restylane* family of products.

# For an authentic you

## The *Restylane*® line of products offers tailored treatments for individual, natural-looking results

Designed to act like the body's own naturally produced hyaluronic acid, the *Restylane* line of products visibly reduces facial wrinkles and folds and adds smooth, silky definition to lips.<sup>2-4</sup>

After



With over 20 million treatments worldwide and counting,<sup>5</sup> the *Restylane* line of products has a rich history of natural-looking results.



*Restylane*,  
for facial wrinkles  
and folds

*Restylane Lyft*,  
for lifted cheeks and  
smoothed wrinkles

*Restylane Silk*,  
for lips and the  
lines around them

Actual, unpaid *Restylane* patient. Individual results may vary. Photos have not been retouched.

# A natural look that lasts



## **Restylane®** has a reputation for enduring results

Widely used for over 2 decades, *Restylane* has helped people all over the world achieve a natural look they love. Adding volume and fullness to the skin, it corrects moderate to severe facial wrinkles and folds, such as the lines from your nose to the corners of your mouth.<sup>2</sup>

*Restylane* offers long-lasting results in nasolabial folds for over 18 months<sup>2\*</sup>—working to provide a smooth, natural-looking appearance that often begs the question: “Have you been on vacation?”

**95%**

of patients still saw improvement 1 year after treatment with *Restylane*<sup>2</sup>

\*In a clinical study treating the lines from the nose to the corners of the mouth (laugh lines), *Restylane* maintained improvement for up to 18 months after initial treatment, when each line received a follow-up treatment at 4.5 or 9 months.

## Real-life results with *Restylane*

Before



After



Victoria

Age 50  
3 mL *Restylane* in nasolabial folds, marionette lines, and upper lip

Before



After



Jeff

Age 48  
3 mL *Restylane* in nasolabial folds

Use of products in the *Restylane* family at the site of skin sores, pimples, rashes, hives, cysts, or infection should be postponed until healing is complete.

Please see Important Safety Information on pages 10-11 for the *Restylane* family of products.

*Restylane*

Actual, unpaid *Restylane* patients. Individual results may vary. Photos have not been retouched.

# A natural-looking lift



## Give cheeks and lines a lift with *Restylane*<sup>®</sup> *Lyft*

You may be noticing some obvious signs of aging that you hadn't previously seen before. What were once subtle lines and wrinkles around the nose and mouth have started to deepen over time. Cheeks that used to appear full and taut suddenly seem to sag and droop.

*Restylane Lyft* is the first and only FDA-approved filler with the versatility of treating both cheeks and nasolabial folds.<sup>3,6</sup>

72%

of patients treated with *Restylane Lyft* felt their cheeks looked smooth and attractive 12 months after treatment<sup>6</sup>

70%

of patients were still satisfied with how their cheeks looked overall 12 months after treatment<sup>6</sup>

## The lift you've been looking for

Before



After



Basia

Age 43  
2 mL *Restylane Lyft* in midface

Before



After



Carol

Age 50  
1 mL *Restylane Lyft* in midface


The most common adverse events after initial treatment include bruising, redness, swelling, pain, headache, tenderness, and itching.

Please see Important Safety Information on pages 10-11 for the *Restylane* family of products.

*Restylane*  
LYFT

Actual, unpaid *Restylane Lyft* patients. Individual results may vary. Photos have not been retouched.

## Naturally smooth, subtle lips

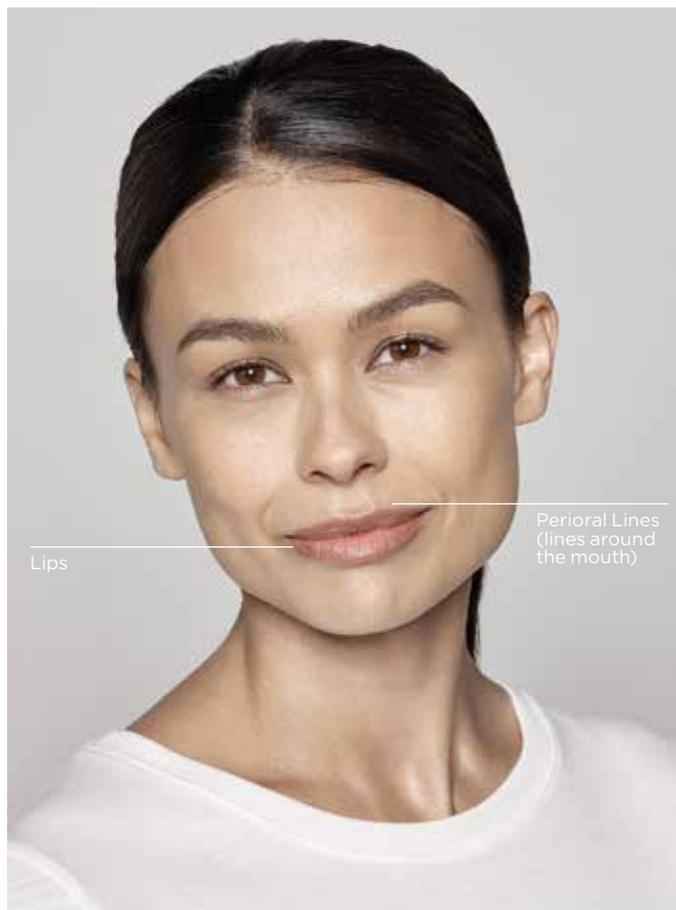

### Treat lips and the lines around them with *Restylane® Silk*

Many people who are comfortable treating other areas of their face are concerned that their lips won't look natural if treated. That's why there's *Restylane Silk*, with its refined smooth gel formulation that allows for subtle, natural-looking results.

*Restylane Silk* is designed for meticulously precise lip definition and line control, giving you the smooth, silky lips you've been longing for. And it's the first and only FDA-approved treatment to specifically address lips and the fine lines around them.<sup>4</sup>

And like other members of its line, *Restylane Silk* is a long-lasting, satisfying treatment.<sup>4</sup>

### Results that lips and lines have been longing for

Before



After



Alanna

Age 32  
2 mL *Restylane Silk* in lips  
and perioral lines

Before



After



Leslie

Age 41  
2 mL *Restylane Silk* in lips

1/2

of patients kept seeing improvement  
24 weeks after treatment<sup>4</sup>

3/4

of patients chose to be treated again<sup>4</sup>

*Restylane Silk* should only be used to be injected into the lips and wrinkles around the lips. It should not be used in other parts of the face and body.

Please see Important Safety Information on pages 10-11 for the *Restylane* family of products.

*Restylane*  
SILK

Actual, unpaid *Restylane Silk* patients. Individual results may vary.  
Photos have not been retouched.

# Important Safety Information and Indication

## Important Safety Information

**Indications:** The *Restylane* family of products includes *Restylane*<sup>®</sup>, *Restylane-L*<sup>®</sup>, *Restylane*<sup>®</sup> *Silk*, *Restylane*<sup>®</sup> *Lyft with Lidocaine*, and *Perlane*<sup>®</sup>. *Restylane*, *Restylane-L*, *Restylane Lyft with Lidocaine*, and *Perlane* are indicated for the correction of moderate to severe facial wrinkles and folds, such as nasolabial folds. *Restylane* and *Restylane-L* are indicated for mid-to-deep dermal implantation. *Perlane* and *Restylane Lyft with Lidocaine* are indicated for implantation into the deep dermis to superficial subcutis. *Restylane Silk* is indicated for submucosal implantation for lip augmentation and dermal implantation for correction of perioral rhytids in patients over the age of 21. *Restylane Lyft with Lidocaine* is also indicated for cheek augmentation and for the correction of age-related midface contour deficiencies in patients over the age of 21. *Restylane* and *Restylane-L* are also indicated for submucosal implantation for lip augmentation in patients over the age of 21.

Products in the *Restylane* family contain traces of gram-positive bacterial protein and are contraindicated for patients with allergies to such material or in patients with severe allergies that have required in-hospital treatment. These products should not be used by patients with bleeding disorders or by pregnant or breastfeeding women. *Restylane* and *Restylane-L* for lip enhancement and *Restylane Silk* should not be used by people under 22 years. *Restylane-L*, *Restylane Silk* and *Restylane Lyft with Lidocaine* should not be used by anyone with a known allergy to lidocaine. Products should not be injected anywhere except the dermis, superficial subcutis (*Perlane* and *Restylane Lyft with Lidocaine* only), or lip submucosa (*Restylane*, *Restylane-L*, and *Restylane Silk* only).

Use of products in the *Restylane* family at the site of skin sores, pimples, rashes, hives, cysts, or infection should be postponed until healing is complete. The most commonly observed side effects are swelling, redness, pain, bruising, headache, tenderness, and itching at the injection site. These are typically mild in severity and typically resolve in less than 7 days. Serious but rare side effects include delayed onset infections, recurrence of herpetic eruptions, and superficial necrosis at the injection site. Do not implant into blood vessels. Use with caution in patients recently treated with anticoagulant or platelet inhibitors to avoid bleeding and bruising.

The *Restylane* family of products is available only through a licensed practitioner. Complete Instructions for Use for *Restylane*, *Restylane-L*, *Restylane Silk*, and *Perlane* are available at [www.RestylaneUSA.com](http://www.RestylaneUSA.com). Complete Instructions for Use for *Restylane Lyft with Lidocaine* is available at [www.RestylaneLyft.com](http://www.RestylaneLyft.com)

**References:** 1. Papakonstantinou E, Roth M, Karakiulakis G. Hyaluronic acid: 2. *Restylane*. Instructions for Use. Galderma Laboratories, L.P. 3. *Restylane Lyft*. Instructions for Use. Galderma Laboratories, L.P. 4. *Restylane Silk*. Instructions for Use. Galderma Laboratories, L.P. 5. Data on file. Galderma Laboratories, L.P. 6. *Restylane Lyft*. Clinical Study Report. Galderma Laboratories, L.P.

The Restylane logo is written in a stylized, cursive script font.

Get the natural-looking results you desire



**The *Restylane*® line of products—tailoring to your individual needs**

- *Restylane*, for facial wrinkles and folds<sup>2</sup>
- *Restylane Lyft*, for cheeks and wrinkles<sup>3</sup>
- *Restylane Silk*, for lips and the lines around them<sup>4</sup>

Talk to your specialist today

*Restylane* *Restylane* *Restylane*  
LYFT SILK



Join ASPIRE Galderma Rewards for instant access to facial aesthetic products, exclusive discounts and more. [ASPIRErewards.com](http://ASPIRErewards.com)